

CURLIN[®] 6000 Ambulatory Infusion Pump

BY MOOG[®]

Worksheet for Non-Library Programming

CONTINUOUS THERAPY

Refer to the CURLIN 6000 User Manual for complete programming instructions.

PATIENT:	PUMP SERIAL #:
----------	----------------

PROGRAMMING / SETUP SEQUENCE

NOTE: Use the **↑ (UP)** or **↓ (DOWN) ARROW** keys to move highlighter and the numeric keys for data entry. Use the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

1. Turn the pump on and wait for the self-test to complete, note the power source with power graph on the screen.

2. Highlight **PROGRAM** from the SELECT menu and press **YES**

NOTE: If the following screen displays "NEW PROGRAM", the PREVIOUS RX has not been cleared. Refer to the last section of the "CURLIN 6000 TRAINING & QUICK REFERENCE GUIDE FOR BETWEEN PATIENT USE" to clear all Previous RX prior to new patient use.

3. Highlight **CONTINUOUS** from the therapy selection menu and press **YES**.

NOTE: If any on-screen alerts appear, the Previous RX has not been cleared. Refer to the last section of the "CURLIN 6000 TRAINING & QUICK REFERENCE GUIDE FOR BETWEEN PATIENT USE" to clear all Previous RX prior to new patient use.

4. With **Unidentified** highlighted, press **YES** (this indicates Manual Programming Mode)

5. **UNITS:** Press **YES** if units desired is indicated (choices are mg, mcg or mL). To change units press **NO** until desired unit is reached, then press **YES** to confirm.

UNITS:	
--------	--

CONCEN: If mg or mcg selected, enter the concentration and press **YES**

CONCEN:	
---------	--

6. **DELAY:** Press **YES** to leave the delayed start feature off, press **NO** to set a delay then, press **YES** to ON:

DELAY:	
--------	--

TIME: Enter time infusion to begin, press **YES**

TIME:	
-------	--

DATE: Enter date infusion to begin, press **YES**

DATE:	
-------	--

ACCEPT?: Press **YES**

7. **medLMTS:** Press **YES** to leave the medication limits Off, press **NO** to set medication limits:

MAX RATE: Enter the upper limit of the infusion rate, press **YES** to confirm

MAX RATE:	
-----------	--

ACCEPT? Press **YES**

8. At **NEXT?** Press **YES** to continue



tel: 801.264.1001
toll free: 800.970.2337
www.curlinpump.com

MOOG[®] Enhancing Healthcare.
Enriching Lives.

9. **BAG VOL:** Enter bag volume, press **YES**. (Bag volume should include overfill volume if pump priming and/or a KVO is programmed.)

BAG VOL:	<input type="text"/>
----------	----------------------

NOTE: To force an auto-calculation, enter any 2 of the following 3 entries: **Amt TBI**, **RATE** or **TIME**. The unknown field will auto-calculate. Bypass the unknown field by using the DOWN ARROW key. Enter the known fields pressing **YES** to confirm each. The unknown field will auto-calculate. Press **YES** to confirm this field.

10. **Amt TBI:** Enter the amount to be infused, press **YES**

Amt TBI:	<input type="text"/>
----------	----------------------

11. **RATE:** Enter the infusion rate, press **YES**

RATE:	<input type="text"/>
-------	----------------------

12. **TIME:** Enter the amount of time for the infusion, press **YES**

TIME:	<input type="text"/>
-------	----------------------

13. **KVO RATE:** Enter KVO (Keep Vein Open) rate, then press **YES** (0 to 10 ml/hr)

KVO RATE:	<input type="text"/>
-----------	----------------------

14. **DONE?:** Press **YES** to accept

When at the **Run to Start Screen**, press the **HELP/OPTIONS** key to set the Lock Level (security level), and to change the Air-In-Line sensitivity and the Down Occlusion sensitivity from the default settings when the therapy requires these changes:

Lock: Press **YES** to accept or **NO** to change Lock setting, press **YES** to accept new setting.

Lock	<input type="text"/>
------	----------------------

DN Occlu: Press **YES** to accept (default setting is LOW) or press **NO** to change then press **YES** to accept.

DN Occlu:	<input type="text"/>
-----------	----------------------

AIR SENS: Press **YES** to accept (default setting is 0.5 mL) or **NO** to change then press **YES** to accept.

AIR SENS:	<input type="text"/>
-----------	----------------------

Press **YES** at each field to advance to the next field. Press **YES** to **ACCEPT OPT** at the end of the menu.

PROGRAMMED BY:	DATE:
CHECKED BY:	DATE:



tel: 801.264.1001
toll free: 800.970.2337
www.curlinpump.com

MOOG® Enhancing Healthcare.
Enriching Lives.

CURLIN® is a registered trademark of Curlin Medical, Inc. Moog® is a registered trademark of Moog, Inc. © 2018 Moog, Inc. Moog Medical, 4314 Zevex Park Lane, Salt Lake City, UT 84123 USA. LS-61473 Rev. A