

CURLIN[®] 6000 Ambulatory Infusion Pump

BY MOOG[®]

Worksheet for Non-Library Programming

PCA THERAPY

Refer to the CURLIN 6000 User Manual for complete programming instructions.

PATIENT:	PUMP SERIAL #:
----------	----------------

PROGRAMMING / SETUP SEQUENCE

NOTE: Use the **↑ (UP)** or **↓ (DOWN) ARROW** keys to move highlighter and the numeric keys for data entry. Use the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

1. Turn the pump on and wait for the self-test to complete, note the power source with power graph on the screen.

2. Highlight **PROGRAM** from the SELECT menu and press **YES**

NOTE: If the following screen displays "NEW PROGRAM", the PREVIOUS RX has not been cleared. Refer to the last section of the "CURLIN 6000 TRAINING & QUICK REFERENCE GUIDE FOR BETWEEN PATIENT USE" to clear all Previous RX prior to new patient use.

3. Highlight **PCA** from the therapy selection menu and press **YES**.

NOTE: If any on-screen alerts appear, the Previous RX has not been cleared. Refer to the last section of the "CURLIN 6000 TRAINING & QUICK REFERENCE GUIDE FOR BETWEEN PATIENT USE" to clear all Previous RX prior to new patient use.

4. With **Unidentified** highlighted, press **YES** (this indicates Manual Programming Mode)

5. **UNITS:** Press **YES** if units desired is indicated (choices are mg, mcg or mL). To change units press **NO** until desired unit is reached, then press **YES** to confirm.

UNITS:	
--------	--

CONCEN: If mg or mcg selected, enter the concentration and press **YES**

CONCEN:	
---------	--

6. **ADMIN Rt:** Press **YES** if administration route desired is indicated (choices are IV, Epidural or SQ). To change ADMIN Rt press **NO** until desired route is reached, then press **YES** to confirm

ADMIN Rt:	
-----------	--

7. **LOAD DOSE:** Enter the loading dose amount if ordered, otherwise leave field at zero, then press YES to confirm.

LOAD DOSE:	
------------	--

NOTE: The load dose is a one-time dose at the start of the infusion. If the pump is turned off before the infusion begins, the loading dose is dropped.



tel: 801.264.1001
toll free: 800.970.2337
www.curlinpump.com

MOOG[®] Enhancing Healthcare.
Enriching Lives.

8. **medLMTS:** Press **YES** to leave medication limits off, press **NO** to set medication limits:

MAX RATE: Enter the upper limit of the basal rate, press **YES**

Δ RATE: Enter the incremental rate change allowed (optional), press **YES**

Δ TIME: Enter the incremental time between allowed rate changes (optional), press **YES**

Max BOLUS: Enter the upper limit of the patient bolus amount, press **YES**

Min INT: Enter the minimum amount of time between patient bolus doses, press **YES**

BOLS/hr: Enter the maximum # of patient bolus doses per hour, press **YES**

ACCEPT? Press **YES**

MAX RATE:	
Δ RATE:	
Δ TIME:	
Max BOLUS:	
Min INT:	
# BOLS/hr:	

9. At **NEXT?** Press **YES** to continue

10. **BAG VOL:** Enter bag volume, press **YES**.

BAG VOL:	
----------	--

11. **BASL RATE:** Enter basal (continuous) rate, press **YES**

BASL RATE:	
------------	--

12. **Pt BOLUS:** Enter patient bolus amount (demand dose), press **YES**

Pt BOLUS:	
-----------	--

13. **BOLUS INT:** Enter the interval between patient bolus doses (in minutes), press **YES**

BOLUS INT:	
------------	--

14. **# BOLS/hr:** Enter the number of patient bolus doses per hour, press **YES**

# BOLS/hr:	
------------	--

15. **DONE?:** Press **YES** to accept

When at the **Run to Start Screen**, press the **HELP/OPTIONS** key to set the Lock Level (security level), and to change the Air-In-Line sensitivity and the Down Occlusion sensitivity from the default settings when the therapy requires these changes:

Lock: Press **YES** to accept or **NO** to change Lock setting, press **YES** to accept new setting.

Lock	
------	--

DN Occlu: Press **YES** to accept (default setting is HIGH) or press **NO** to change then press **YES** to accept.

DN Occlu:	
-----------	--

AIR SENS: Press **YES** to accept (default setting is 0.5 mL) or **NO** to change then press **YES** to accept.

AIR SENS:	
-----------	--

Press **YES** at each field to advance to the next field. Press **YES** to **ACCEPT OPT** at the end of the menu.

PROGRAMMED BY:	DATE:
CHECKED BY:	DATE:



tel: 801.264.1001
 toll free: 800.970.2337
 www.curlinpump.com

MOOG® Enhancing Healthcare.
 Enriching Lives.

CURLIN® is a registered trademark of Curlin Medical, Inc. Moog® is a registered trademark of Moog, Inc. © 2018 Moog, Inc. Moog Medical, 4314 Zevex Park Lane, Salt Lake City, UT 84123 USA. LS-61475 Rev. A